


User Measurements

Size	Small		Medium		Large		X-Large		User's Measurements
User Height	90-110 cm	35.43-43.31"	110-130 cm	43.31-51.18"	130-150 cm	51.18-59.10"	150-170 cm	59.10-67"	
A Seat Depth	26cm	10.24"	32cm	12.6"	40cm	15.75"	45cm	17.72"	
B Seat Width at Back	20cm	7.87"	23cm	9.1"	27cm	10.63"	30cm	11.81"	
C Seat Width at Front	27cm	10.63"	30.5cm	12"	37cm	14.56"	40cm	15.75"	
D Back Height	60cm	23.62"	69cm	27.17"	74cm	29.13"	80cm	31.5"	
E Seat Front Height	20cm	7.87"	23cm	9"	26cm	10.24"	29cm	11.42"	
F Arm Support Height	29cm	11.42"	35cm	13.78"	39cm	15.35"	43cm	16.93"	
G Total Width of Chair	46cm	18.11"	49cm	19.29"	57cm	22.44"	60cm	23.62"	
H Total Height of Chair	55cm	21.65"	61cm	24"	66cm	25.98"	71cm	27.95"	
I Total Depth of Chair	60cm	23.62"	71cm	28"	82.5cm	32.48"	91cm	35.83"	
J Tilt Wedge Length	47cm	18.5"	57cm	22.44"	67cm	26.38"	74cm	29.13"	
K Tilt Wedge Height	10cm	3.94"	12cm	4.72"	14cm	5.51"	17cm	6.69"	
Weight Limit	50kg	110lb	70kg	154lb	100kg	220lb	130kg	278lb	
Classic Version, Red Liner	3770-0011-022	<input type="radio"/>	3770-0012-022	<input type="radio"/>	3770-0013-022	<input type="radio"/>	3770-0014-022	<input type="radio"/>	User's Name
Classic Version, Green Liner	3770-0011-055	<input type="radio"/>	3770-0012-055	<input type="radio"/>	3770-0013-055	<input type="radio"/>	3770-0014-055	<input type="radio"/>	
Classic Version, Blue Liner	3770-0011-066	<input type="radio"/>	3770-0012-066	<input type="radio"/>	3770-0013-066	<input type="radio"/>	3770-0014-066	<input type="radio"/>	
Wipe-Down Version, Red Liner	3770-0021-022	<input type="radio"/>	3770-0022-022	<input type="radio"/>	3770-0023-022	<input type="radio"/>	3770-0024-022	<input type="radio"/>	
Wipe-Down Version, Green Liner	3770-0021-055	<input type="radio"/>	3770-0022-055	<input type="radio"/>	3770-0023-055	<input type="radio"/>	3770-0024-055	<input type="radio"/>	
Wipe-Down Version, Blue Liner	3770-0021-066	<input type="radio"/>	3770-0022-066	<input type="radio"/>	3770-0023-066	<input type="radio"/>	3770-0024-066	<input type="radio"/>	

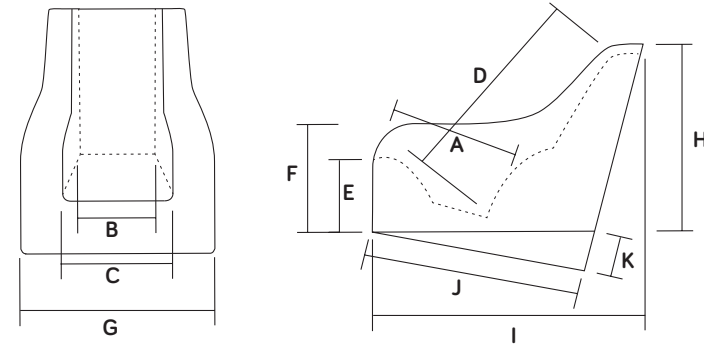

Tray Table

Small	Large
Fits Gravity Chair size 1 & 2: 3770-7101-000 <input type="radio"/>	Fits Gravity Chair size 3 & 4: 3770-7102-000 <input type="radio"/>



Mobile Base

Size 1	Size 2	Size 3	Size 4
3780-8701-000 <input type="radio"/>	3780-8702-000 <input type="radio"/>	3780-8703-000 <input type="radio"/>	3780-8704-000 <input type="radio"/>



Supportive headrest

Occipital Head Support	Standard Lateral Head Support	Extended Lateral Head Support
i. Black Spacer fabric: 3780-4101-099 <input type="radio"/> ii. Spextex wipe-down fabric: 3780-4101-209 <input type="radio"/>	i. Black Spacer fabric: 3780-4201-099 <input type="radio"/> ii. Spextex wipe-down fabric: 3780-4201-209 <input type="radio"/>	i. Black Spacer fabric: 3780-4301-099 <input type="radio"/> ii. Spextex wipe-down fabric: 3780-4301-209 <input type="radio"/>



medifab®

32 Detroit Drive
Rolleston 7675
New Zealand

+64 3 307 9790

solutions@medifab.com

medifab.com



medifab® | Gravity Chair



medifab®

Gravity based postural management

medifab® Gravity Chair

More than just
a comfortable
seat...



Supportive foam positioning system to augment postural control for a wide range of individuals. The Gravity Chair follows the principle 'Tension and Relaxation' whereby it:

1. Provides alternative seating
2. Serves the therapeutical aim of stable positioning

CENTRE OF GRAVITY
is very low to offer a stable seating position

Raised armrests **'BOLSTER' THE USER Laterally.** This high armrest position facilitates free breathing to enable increased blood oxygen.

Seating configuration **UTILISES GRAVITY** to 'hold' the individual comfortably

Very light, which enables it to be used almost anywhere!



Using the wedge accessory, the head and body position can be stabilised with tilt (including sideways tilt), for patients such as those with Scoliosis.

Client group

- Patients who can only lie down learn to be able to sit in the Gravity Chair.
- With patients who spasm, the Gravity Chair absorbs tension. After the spasm, the chair brings the user back into the original position.
- Patients with perception deficit or ADHD get close body contact in the Gravity Chair which raises self-awareness. This helps the user to stay calm.
- The Gravity Chair can create an optimal solution for many occupational therapy requirements.
- Palliative care.



Accessories

Mobile Base
1 3780-8701-000
2 3780-8702-000
3 3780-8703-000
4 3780-8704-000



Therapy Tray Table
Small 3770-7101-000
Large 3770-7102-000
Wide 3780-7103-000



Occipital Head Support
i. Black Spacer fabric: 3780-4101-099
ii. Spextex wipe-down fabric: 3780-4101-209



Standard Lateral Head Support
i. Black Spacer fabric: 3780-4201-099
ii. Spextex wipe-down fabric: 3780-4201-209



Extended Lateral Head Support
i. Black Spacer fabric: 3780-4301-099
ii. Spextex wipe-down fabric: 3780-4301-209



Which cover is best for me?

Classic Gravity Chair

- Air-breathable material on internal contact surfaces.
- Liner can be removed and washed.
- A wipe-down material over the majority of the chair for easy cleaning.



Wipe-Down Gravity Chair

- Full wipe-down including internal contact surfaces where the user is seated.
- A great option for those with continence/reflux issues, or for any messy eaters.
- Easy wipe-down cleaning.



Colors

Choose from three different colors for the inner material on either version of the Gravity chair. The chair itself is a light neutral color.

